Uso previsto

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Intended use

Dental impression materials on the basis of vinylpolysiloxane, mixing ratio A:B = 1:1.

Indications for use

Dynax® dental impression material is intended to be placed on a preformed impression tray and to reproduce the structure of a patients teeth and gums.

Properties Dynax® putty, Dynax® heavy body, Dynax® mono, Dynax® clear and Dynax® light:

- scannable surface
- hydrophilic surface characteristics
- high tensile strength
- highest precision and dimensional stability
- high elastic recovery
- long processing times
- short intraoral curing

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Dynax® light is a light bodied correction material with high detail precision for the sandwich, correction and double mixing technology. Combinable with **Dynax® putty**, **Dynax® heavy body**, **Dynax® mono** and **Dynax® clear**.

Dynax® mono is a medium bodied precision impression material for the monophasic impression.

Dynax® clear is a clear transparent, medium bodied precision impression material for the monophasic impression, especially for the implant impression.

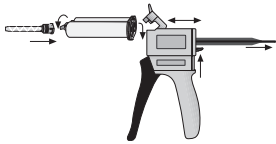
Impression tray: Preparation and adhesives

All standard (in metal or hard plastics) and / or individual impression trays are applicable corresponding to the impression technique.

To improve the adhesion of the impression material to the metal or plastic trays, a tray adhesive for A-silicones should be applied.

Processing 50 ml double cartridge

The impression material is automatically dosed out of the double cartridge in a mixing ratio of 1:1. Put the double cartridge into the injector. Remove the cap of the cartridge and dispense a bit of the material until both components flow equally. Now, attach the mixing canula onto the double cartridge and start dispensing the material into the prepared impression tray.

Processing tubs

Mixing: Take both components A and B out of the tubs in the ratio 1: 1 by using the supplied trays and mix both components quickly by hand until a homogeneous color arises. The A and B components of a packaging must only be used in this combination. Tubs should be closed well after use.

Keep away from children! For dental use only!

Storage conditions

At room temperature, i. e. 18°C–28°C (64°F–82°F).

Lot number / Expiry date

The lot number and the expiry date are indicated on the external packaging as well as on each double cartridge or tub. In case of claims please always indicate the lot number of the product. Do not use the product after expiration.

Please find processing times in „technical data“ overview.

Disinfecting

For disinfecting the impression should be soaked in disinfecting solution. The usage of a 2 % glutaraldehyde solution is recommended. Please make sure that the used disinfecting solution is compatible for being used with addition-vulcanising silicones. After removing the impression from the mouth it has to be washed up with running water for 15 seconds. Afterwards disinfect and then rinse with running water for 15 seconds in order to remove the disinfecting solution. The disinfection of the injector can be carried out with a common immersion disinfection.

Casting

After disinfecting dry the impression and store it at normal room temperature. The impression should not be cast any sooner than 30 min. after removal from the mouth. Using special purpose hard plaster (ISO 6873, type 3) respectively an ultra hard plaster for the forms (ISO 6873, type 4) is recommended.

Important advice

- **Dynax®** products are not to be used in combination with condensation-vulcanising impression materials.
- When using double cartridges, the mixing canula should remain on the cartridge as sealing.
- The setting times can be delayed by cool storage or shortened by warmer storage.
- Residues of retraction aids have to be removed by use of water spray.
- The polymerisation, i. e. the setting of addition-vulcanising material can be interrupted by latex gloves, cremes, cleaning agent, resins etc. Therefore, we recommend to wear standard gloves made of polyethylene. Mixed material is chemically persistent. Spots onto clothing cannot be removed.

Keep away from children!

For dental use only!

Storage conditions

At room temperature, i. e. 18°C–28°C (64°F–82°F).

Lot number / Expiry date

The lot number and the expiry date are indicated on the external packaging as well as on each double cartridge or tub. In case of claims please always indicate the lot number of the product. Do not use the product after expiration.

This product was developed for dental use and has to be applied according to the instructions. In the case of damages resulting from ulterior or improper use, the manufacturer assumes no liability. Furthermore the user is bound to check the product according to suitability and possible use in self dependence before using it subject to its purpose, especially in case the purpose is not mentioned in the instructions.

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